



Daily Startup Checklist

TASK	ACTION	
1. Verify machine is on.	The DSD-201 should not be unplugged during normal use.	
2. Open incoming water line valve.	Confirm that the static water pressure is at least 35-40 PSI (2.4-2.75 BAR)	
3. Check the disinfectant level.	Add disinfectant as needed, to avoid alarms and to properly disinfect endoscope.	
4. Check the disinfectant temperature.	If a heated disinfectant is being used, check its temperature by using SET UP 13.	
5. Check the expiration date of the disinfectant.	Do not use disinfectant beyond the manufacturer's recommended re-use life, even if the Minimum Recommended Concentration (MRC) is still achieved.	
6. Test the potency of the disinfectant in both reservoirs, prior to each use.	Use a test strip recommended by the disinfectant manufacturer. Replace disinfectant if it does not meet MRC.	
7. Check the alcohol level.	Add alcohol, if necessary, to avoid alarms.	
8. Check the time/date and program number selected on the display.	Adjust using the SET UP function, if needed.	
9. Check the printer paper and ribbon level.	Change, if needed.	
10. Check filters logs.	Confirm all filters are changed at required intervals:	
	1 and .45 micron External Water Filters	Replace when water pressure of 40 PSI cannot be maintained or every 3 months.
	0.2 micron Internal Water Filter	Replace filter and disinfect filter and water lines at least every 6 months.
	Disinfectant Filter	Replace during every disinfectant change
	Air Filter	Replace every 3 months.
11. Pull air tank ring.	To remove any condensation in air tank.	

Daily Shutdown Checklist

TASK	ACTION
1. Close incoming water line valve.	To protect facility and filter housings from potential leaks and associated water damage.
2. Wipe down basins and basin covers	Do not use alcohol-based products on the DSD-201, as they cause drying, crazing and cracking.